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Abspülen

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Aufsetzen

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Instructions for Use for Soft and Rigid Gas Permeable Contact Lenses

Information sheet

General notes

Please read the Instructions for Use before using your contact lenses. If you have any questions about wearing contact lenses, please contact your contact lens specialist.

Follow the recommendations of your contact lens specialist, the information on the labels of the contact lens care products and all the instructions in this brochure.

Indications and Intended Use/Purpose

Optimisation of visual acuity in persons with medically healthy eyes, who have ametropia or aphakia. Protection of sensitive cornea. Correction of the following defective vision:

- short-sightedness (myopia) including myopia management,
- long-sightedness (hyperopia),
- astigmatism,
- age-related long-sightedness (presbyopia),
- keratoconus and
- combinations thereof.

Clinical Benefit and Performance Characteristic

Improving visual acuity while wearing the contact lenses.

Target User Group/Intended Users

For the correction of ametropia and optimisation of vision in infants, children, adolescents and adults.

Wearing time, period of use, checks

Your contact lenses are suitable for daily use. Unless they are rigid gas permeable contact lenses for orthokeratology, they are designed to be suitable for daytime wear, i.e. for use during normal waking hours. The maximum daily wearing time recommended by your contact lens specialist must not be exceeded.

The replacement schedule of your contact lenses depends on various factors, such as the contact lens material, handling, care, mechanical stress and the way you wear them. Your contact lens specialist will inform you when to replace your contact lenses during your regular check-up. If the manufacturer prescribes replacement intervals,

these must not be exceeded. If you do not wear your contact lenses for an extended period of time, they may need to be disinfected again before you next wear them. Please refer to the Instructions for Use of your lens care product.

Handling instructions

Wash and dry your hands thoroughly before touching your contact lenses. Rinse the contact lenses only with saline solution or with the care product intended for this purpose. If a contact lens is dropped, be sure to clean it with a suitable rinsing solution before continuing with the actual cleaning process.

Always check that the contact lenses are intact before putting them in.

Put in or take out your contact lenses prior to applying or removing cosmetics. Ensure that any cosmetics that you use are compatible with contact lenses. Do not touch your contact lenses if you have recently applied cream to your hands. Avoid long fingernails, as bacteria can collect under them and they can also cause injury to your eyes or damage the contact lenses.

Avoid rubbing your eyes while wearing contact lenses. The contact lenses could slip out of place or fall out. There is also a risk of contamination.

If a contact lens sticks to the eye, we recommend putting a few drops of saline solution directly into the affected eye. After a few moments, it should then be possible to move the contact lenses freely in the eye. If this does not happen, you should get in touch with your contact lens specialist.

Contact lenses must not be used by more than one person. To avoid mixing up your contact lenses with those of someone else, we recommend always storing them in the same place, and clearly differentiating the container that they are kept in (marking it, or using colour coding, etc.). Should contact lenses become mixed up, you will most probably notice it due to blurred vision and also because wearing them feels different. Due to possible cross-contamination, repeated disinfection or replacement is recommended.

Special handling notes for third parties

For users who are not or only insufficiently able to use the products according to the Instructions for Use, the correct use must be ensured by third parties.

Warnings

Avoid contact with any liquids that are not intended for the care of contact lenses. Take appropriate protective measures if you wear your lenses during water sports, for example, or in the presence of toxic or irritating vapours or occupational hazards. It is recommended that you remove your contact lenses before bathing/showering, and also in the sauna.

Store and transport your contact lenses at normal room temperature between 15 and 25°C. Briefly exceeding or falling below the specified temperatures has no effect on the performance and safety of the products. Never store contact lenses in places where they are exposed to intense sunlight (e.g. in a car). The temperatures which occur in such situations (over 50°C) may cause irreversible deformation of the contact lenses.

Special environmental conditions, such as aerosols, liquids, dust, dirt, steam, sprays, heat and cold, may impair the wearing of contact lenses. In these situations, ask your contact lens specialist in advance.

Contraindications

Do not use the contact lenses in the following cases:

- If there is any irritation, inflammation or infection of the eye, or if you sustain an injury to the eye
- In the event of altered corneal sensitivity or a lack of tears
- In the case of a general disease associated with a weakened immune system
- Allergies
- If you take any medications, and
- Only in consultation with your contact lens specialist if you have an eyelid defect of any kind.

Side effects

Remove your contact lens from your eye immediately if you experience the following or similar symptoms:

- Reduced wearing comfort
- Excessive tears
- Increased redness of the eyes
- Stinging, burning, pain in the eyes
- Extreme dryness of the eyes
- Excessive sensitivity to light and glare
- Sudden visual impairment, foggy vision
- Allergic reactions
- Formation of secretion on the eye

Examine your contact lenses carefully for any defects or contamination before putting them in. A damaged contact lens must not be put back in the eye again. If the contact lens is contaminated, clean, rinse and disinfect it thoroughly before putting it back in. If the problem persists, remove the contact lens from the eye again and consult your contact lens specialist.

Interactions with other household, medicinal or healthcare products

Never use liquids (including tap water) that are not intended for the care of your contact lenses. Permanent damage to the eyes could result.

Medications taken over an extended period of time can cause a systemic change in the tear film and thus affect your tolerance of contact lenses. Inform your contact lens specialist about the medication you are taking. Some medications or nutritional supplements may, in individual cases, lead to discolouration, changes in shape and/or eye irritation.

When using eye drops, you should observe the corresponding indications in the package inserts of these products.

Special notes

Always keep your contact lenses in a suitable closed container and out of the reach of children. There is a risk of suffocation if contact lenses are swallowed. Contact lenses always need to be carefully cleaned and cared for according to the respective instructions. If you notice any damage, scratches or deposits on the contact lens or if a soft

contact lens has dried out, do not continue to use it. Consult your contact lens specialist.

Please only dispose of your contact lenses and accessories in the general household waste. They are not suitable to be disposed of in the organic waste or rinsed down the toilet or the sink.

Handling soft contact lenses

(product groups: Invispa, Individual)

All soft contact lenses produced by Galifa Contactlinsen AG are supplied sterile and are individually stored in a container secured with a tamper-proof closure. The heat-generated sterility of the product lasts for up to two years from the date of manufacture (MFD). The expiry date (EXP) specifies the date until which sterility is guaranteed. Up to this point, the contact lenses can be taken directly out of the solution and put into the eye. Only use contact lenses from undamaged primary packaging, with the tamper-proof closure intact.

Handling soft contact lenses correctly

Putting in soft contact lenses

1. Make sure that the contact lenses are not turned inside out. For this purpose, lay them on the tip of the index finger and verify the proper shape (see Figure 1).

Figure 1

2. Look into a mirror with both eyes. Place the contact lens on the tip of the index finger and pull down the lower eyelid with the middle finger of the same hand. Pull the upper lid upwards with the middle and/or ring finger of the other hand. Place the contact lens in the centre of the cornea, without applying any pressure, and carefully close the eye (see Figure 2).

Figure 2

3. Verify in the mirror that the lens is correctly positioned, and repeat the process with the other eye.

Removing soft contact lenses

1. Pull the lower lid downwards with the middle finger, and the upper lid upwards with the index finger of the other hand (see Figure 3). Push the contact lens downwards a little, using the index finger of the first hand, onto the white conjunctival region of the eye. Look slightly upwards while doing this.
2. Carefully fold the contact lens with the thumb and index finger, and remove it from the eye (see Figure 3).

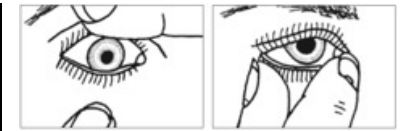
Figure 3

Special notes on handling soft contact lenses

Never store your soft contact lenses dry. The contact lens will dry out, and can no longer be used.

Handling rigid gas permeable contact lenses

(product groups: Modula, Rose K, MK Pro, Luna)

All rigid gas permeable contact lenses of Galifa Contactlinsen AG are supplied individually, disinfected, and preserved in a storage solution. The disinfectant effect is maintained for up to four weeks, as from the date of manufacture. Within this period of time, the contact lens can be taken directly out of the solution and put into the eye.

Once the expiry date has been exceeded, the lenses need to be cleaned and disinfected again, using a suitable contact lens care product, prior to being put in the eye. Only use contact lenses from undamaged primary packaging.

Handling rigid gas permeable contact lenses correctly

Putting in rigid gas permeable contact lenses

1. Look into a mirror with both eyes. Place the contact lens on the tip of the index finger and pull down the lower eyelid with the middle finger of the same hand. Pull the upper lid upwards with the middle and/or ring finger of the other hand. Place the contact lens in the centre of the cornea, without applying any pressure, and carefully close the eye (see Figure 4).
2. Verify in the mirror that the lens is correctly positioned. If the contact lens is not centred on the cornea: carefully remove it from the eye with a suitable contact lens remover and try to place it correctly again. Repeat the procedure for the other side (see Figure 4Abbildung 4).

Figure 4

Removing rigid gas permeable contact lenses

The «pulling the lids to the side» method:

1. Bend your head over a horizontal mirror. Open the eye wide enough so that the rigid gas permeable contact lens is no longer covered by the lids.
2. Place the tip of your finger vertically in the outer corner of the eye, without touching the eye, and pull the edges of both lids firmly in the direction of the ear. The contact lens should then loosen itself from the eye. Let it drop into your opened hand (see Figure 5).

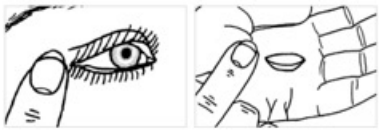
Figure 5

The alternative method of «taking the contact lens out with a contact lens remover»:

→ Recommended for large diameter contact lenses and contact lenses for orthokeratology.

1. Before trying to remove the lens, make sure that it is located centrally on the cornea.
2. Hold the contact lens remover between your thumb and index finger, pull the lower lid down with the middle finger of the same hand, and pull the upper lid upwards with the index finger of the other hand. Carefully place the contact lens remover in the centre of the contact lens and remove it with a pulling motion. Take the contact lens with your thumb and index finger and remove it from the side of the contact lens remover in a twisting motion (see Figure 6).

Figure 6

Special notes on handling rigid gas permeable contact lenses

Pick up any contact lens that has been dropped with a moist fingertip or the contact lens remover.

In order to avoid scratches and damage, never drag the contact lenses over the surface of a table or mirror.

When removing the contact lens using the alternative method of «taking the contact lens out with a contact lens remover», make sure that you only touch the contact lens. Otherwise, serious injuries to the eye may result. We strongly recommend the use of a safety contact lens remover.

Special notes on handling rigid gas permeable contact lenses for orthokeratology

Rigid gas permeable contact lenses for orthokeratology (Luna) are intended to be worn while asleep. If the treatment is successful, it is not necessary to wear an additional visual aid during waking hours. Should you, however, notice a loss in visual acuity in the evening, you can use your rigid gas permeable contact lenses for orthokeratology. However, the lenses may be less comfortable during this time.

In the initial phase of orthokeratology therapy, a gradual loss of visual acuity during the day can be observed as the Ortho-K effect subsides. Your contact lens specialist will inform you whether it is necessary to wear an additional visual aid for driving a vehicle, especially in the evening.

The period of time after which rigid gas permeable contact lenses for orthokeratology need to be exchanged is reduced to a maximum of 12 months.

If you rub your eyes without noticing it while you are asleep, the contact lenses may, under certain circumstances, be displaced or fall out.

Please check your visual acuity every time you take them out in the morning. If you experience blurring or have any other doubts about the visual quality that you are experiencing, please contact your contact lens specialist – only the contact lens specialist can certify your fitness to drive.

Rinse and fill contact lenses with preservative-free saline solution before putting them in your eye to ensure ideal efficiency and hydration under the contact lens. We strongly discourage the use of contact lens and re-hydration solutions of higher viscosity.

In rare cases, the contact lens may become stuck to your eye overnight. If there is no visible movement of the contact lens in the morning before taking it out, or if there is greater resistance than usual, free the contact lens by applying light pressure with the help of the lower eyelid.

How to properly take care of your contact lenses

Daily cleaning and disinfection of your contact lenses is necessary for safe, comfortable contact lens wearing. In that respect, regularity and care play a critical role. Lack of care may lead to serious damage to the eye.

Please only use care systems, containers and accessories that have been recommended by your contact lens specialist, or that are compatible with the contact lenses concerned. For safe use, adhere very precisely to the instructions for use accompanying the respective care products. Do not use conditioning solution more than once. Do not use any care products where the expiry date has been exceeded. When using hydrogen peroxide-based products, please pay attention to reliable neutralisation, whether by tablet or catalyst. Non-neutralised or insufficiently neutralised hydrogen peroxide solutions may cause serious eye irritation and damage.

- **Clean** your contact lenses after each use in order to thoroughly remove any tear film residues and other foreign bodies, e.g. from cosmetics. Place one or two drops of the cleaning solution on the inside and outside of the contact lens and rub it between the thumb and index finger for at least 20 seconds without applying pressure.
- Then **rinse** the contact lens thoroughly with a rinsing solution and place it in the container filled with fresh conditioning solution.
- The conditioning solution contains disinfecting substances. To ensure that the **disinfection** process is correct and complete, be sure to follow the manufacturer's instructions. Never use conditioning solution more than once.
- **Additional special cleaning products and protein removers:** In some cases, the use of additional surface or protein cleaning products is recommended. Your contact lens specialist will advise you on the best way to clean your contact lenses.

Please get in touch with your contact lens specialist if you cannot remove impurities. Clean your

contact lens container after each use, and replace it regularly.

If you have not worn your contact lenses for a few days, disinfect them again prior to the next use.

⚠ Serious incidents

If there are any serious incidents in connection with your contact lenses, please get in touch with your contact lens specialist. The manufacturer and the competent authority of the Member State where the user is located need to be notified of the incident.

Labelling

The symbols shown in the Instructions for Use and on the labels of the contact lenses are explained in following overview. In addition, the serial number for unique identification is engraved on every contact lens.

	Caution
	Follow the instructions for use
	Authorised representative in the European Community
	EC mark of conformity
	Manufacturer
	Manufacturing date YYYY-MM-DD
	Heat-sterilised, incl. Name and address
	Medical device
	Serial number
	Temperature limits
	Expiration date YYYY-MM-DD
	Single sterile barrier system

Galifa Contactlinsen AG,
Zürcherstrasse 204e, 9014 St.
Gallen, Schweiz
Tel. +41 (0) 71 272 30 00
www.galifa.swiss
info@galifa.swiss

Galifa Contactlinsen GmbH,
Marktstrasse 30, AT-6850
Dornbirn

To be filled in by the contact lens specialist

Your contact lenses

- Soft
- Rigid gas permeable
- Rigid gas permeable for orthokeratology

R L

Product name

SN

Contact lens specialist

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1st day

2nd day

3rd day

4th day

5th day

6th day

Maximum wearing time following the introductory period

Check-ups

Check-ups Recommended wearing time

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